Martha - ALS Detailed view by Bill Weis

Requirements:

1. Martha wants to be able to use voice to control her Joerns Ultra Care UCXT bed.


Solution – High Level:

1. We provided a voice activated controller to control the head up/down and foot up/down functions of her bed.

Solution Details

1 - Martha wants to use voice commands to control her Joerns UltraCare UCXT full electric bed - Martha has a 6 function Joerns UltraCare bed. The controls in this bed are from Linak, and the control box itself is a CB6S which supports Linak's Openbus. By adding a MJB5094001-1323 device, we were able to connect the RJ50 cable from our voice activated bed controller to port 3 on the MJB. The hand control was plugged into OpenBus port 2, and an RJ50 short cable was plugged into OpenBus port 1 and connected to the hand control port on the CB6S Linak control box. We provided Martha with several voice commands that run for specific durations to raise and lower the head and foot end of the bed. For safety reasons we do not provide voice control of the Hi Lo functions, but they are available to the caregiver through the standard hand control.

Schematic on next page

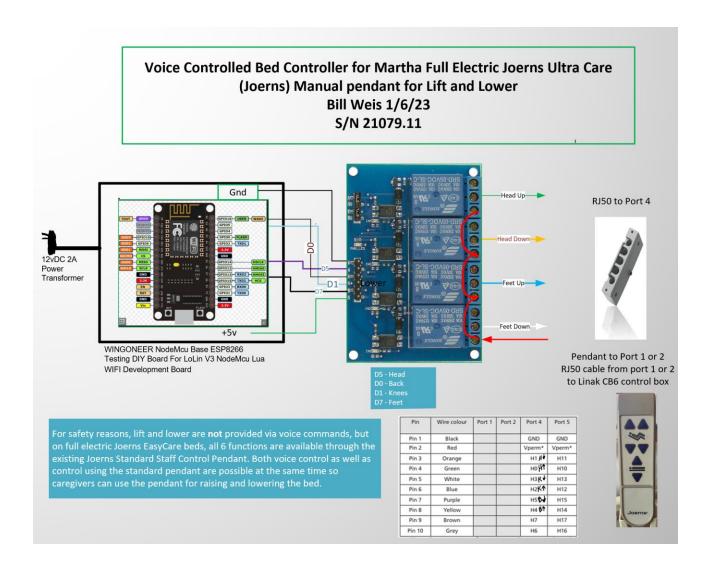

Resources

Google Home getting started

Google Home Help Forum

Google Home Support Phone number for Google Home hardware support = 855-971-9121 (24/7 days a week)

<u>Wemo Support</u> Phone number for Support = 1-844-745-wemo (9366)