Tommy Vilardo - Progressive Multiple Sclerosis

Requirements:

1. Be able to voice control his bed (Invacare Full Electric Homecare Bed model 5410IVC with pendant 1115290). (Tommy's family has already implemented a voice solution for the TV and some lights).


Solution – High Level:

1. Designed a microcontroller-based Voice Activated solution that provided the desired functions of Head Up, Head Down, Feet Up, Feet Down. All four of these functions are available via voice commands as well as using a pendant. Lift and lower functions are available via pendant only.

Details of the Solution

1 – Voice Control his bed – The bed is an Invacare Full Electric Homecare Bed model 5410IVC with pendant 1115290 that is direct wired to the bed. The approach was to have an ESP8266 microcontroller drive 4 mechanical relays which replaced the 1115290 pendant. The 4 functions of Head Up, Head Down, Feet Up, Feet Down are each controlled by one of the 4 relays. We also provide an RJ50 connector on our bed controller to allow for a hand held 1115290 to be added, therefore both voice commands as well as manual use of the pendant will control the bed. The pendant is the only way that bed lift and bed lower functions due to safety reasons.

Our bed controller responds to commands from Alexa as well as Google which provides redundancy as well as giving the customer the opportunity to see which smart speaker understands them better.

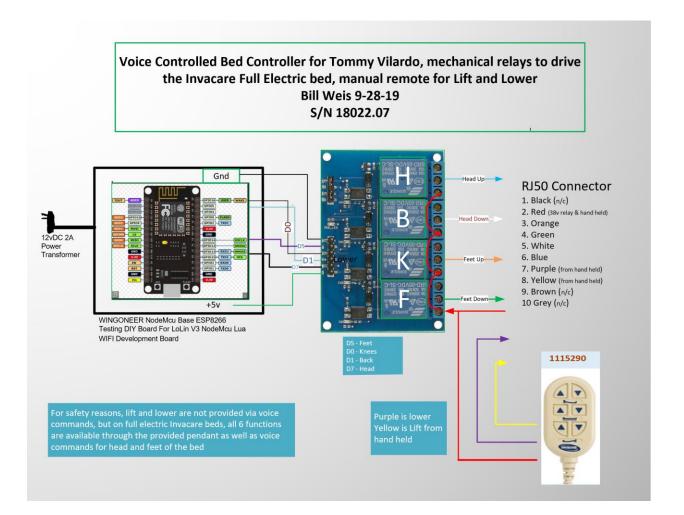

Resources

Amazon Echo

<u>Alexa Support</u> (Contact Support via the Amazon Alexa app - can have them call your number)

Google Home getting started

Google Home Help Forum

Google Home Support Phone number for Google Home hardware support = 855-971-9121 (24/7 days a week)

Logitech Harmony Knowledge Base

<u>Logitech Harmony Support</u> Phone # for Support = 866-601-5644 (M-F 8am to 6pm PST) <u>Lifx</u>

Wemo Support Phone number for Support = 1-844-745-wemo (9366)